Fig. 3 / Rys. 3

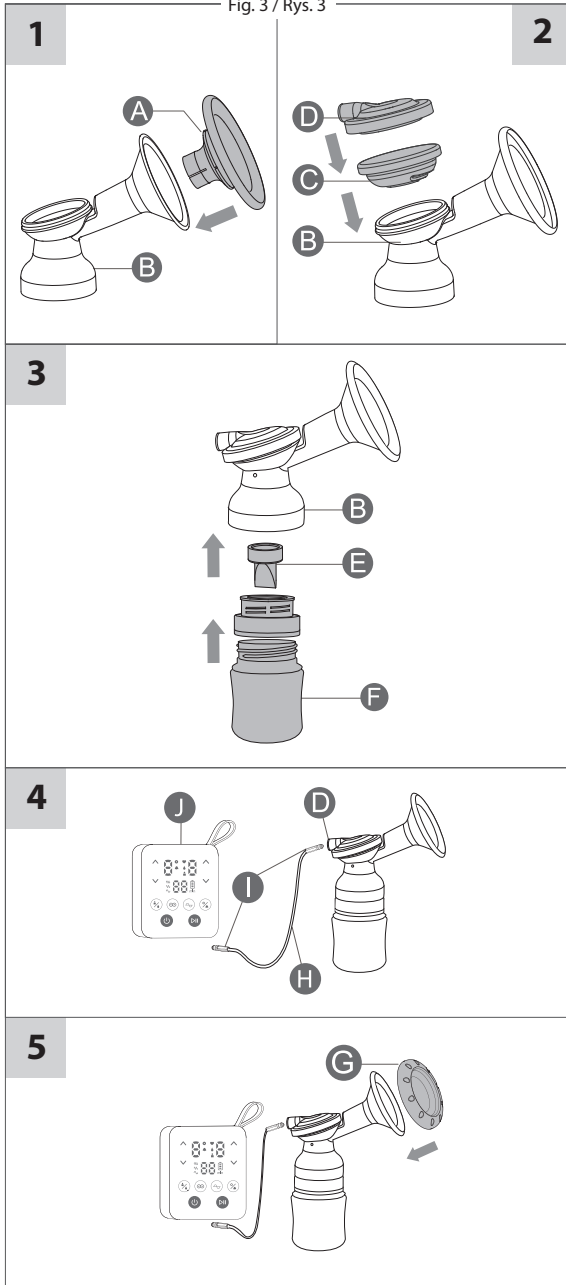
Fig. 4 / Rys. 4

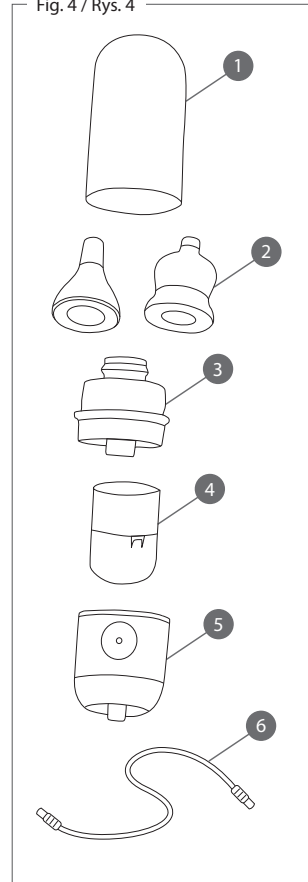
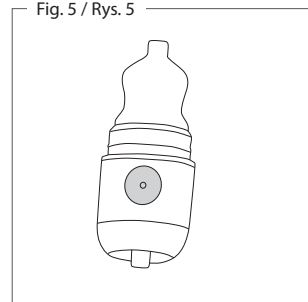
Fig. 5 / Rys. 5

## Double Electric Breast Pump

Model 12/212

### BASIC PRODUCT INFORMATION

This eclectic breast pump (hereinafter referred to as the breast pump) is a product designed for breastfeeding mothers. It allows mothers to express milk for later use, i.e. in the case of separation from their child. The breast pump helps mothers to maintain lactation, plan their absence and ensure milk in case of emergencies such as power cuts, medical treatment, or cracked and sore nipples.

- The device is intended for personal use. Use by more than one person may pose a health risk.
- The device comes with a 24-month manufacturer's warranty. Please keep your proof of purchase of the device.
- Before using, read the user manual and follow the rules of safe use.
- Keep this user manual together with the device.
- The expected lifetime of the product is 5 years. The expected lifetime of silicone components is no more than 2 years. Silicone components may change their colour permanently. Replace silicone components after finding discolouration or after the estimated time of use.

**FOLLOW THE INSTRUCTIONS BELOW STRICTLY. THE MANUFACTURER IS NOT LIABLE FOR DAMAGES RESULTING FROM USE INCONSISTENT WITH THIS USER MANUAL OR IMPROPER USE. ANY DAMAGES ARISING FROM MISUSE CAN NOT BE THE BASIS FOR COMPLAINTS OR REPAIRS UNDER WARRANTY.**

**Making any repairs or modifications to the product by unauthorised persons results in loss of warranty and may result in risks to health and life.**

### GENERAL SAFETY RULES

When using electrical devices, take the following safety precautions.

- **WARNING:** To protect against electric shock, do not immerse the power adapter, the USB power cable or the breast pump body in water or any other liquid. Protect the device from getting wet, and DO NOT operate the device with wet hands.
  - Before you plug in the device for the first time, ensure that the voltage is compatible with your electricity supply by checking the label on the base of the unit. External power supply voltage shall be 100-240 V.
  - To charge the built-in battery, connect the device only to an efficient and surge-protected mains supply.
  - If needed, use only an adequately connected and protected extension cord.
  - If you use extension cord, please remember:
    1. The extension cord must bear the CE marking.
    2. The extension cord cannot bear signs of any damage.
    3. Arrange the cord so that it does not drape over the countertop or tabletop where it can be pulled by children or tripped over.
  - Do not leave the device unattended when it is plugged into a power socket.
  - When removing the plug from the socket, hold the edge of the socket. Never pull the plug out of the socket by pulling on the power cord.
  - Do not place any power cords under furniture or other obstacles.
  - DO NOT put on or near a hot gas or electric stove, or in a preheated oven. Make sure that the power cords are away from hot surfaces, radiators and central heating pipes. Do not leave the device directly in the sun.
  - Always unplug the device from the socket before cleaning.
  - **NOTE!** Do not use any device with a damaged cord or plug. Do not use the device after its failure or any other damage.
  - Never use an incomplete or broken device. Replace damaged or worn parts immediately. Use only replacement parts provided by the manufacturer.
  - Follow the recommendations on the correct use of the device and its accessories.
  - This device is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge unless they have been given supervision or instruction concerning the use of the appliance by their guardian.
  - Close supervision is necessary when any appliance is used by a disabled person or near children.
  - ALWAYS keep out of the reach of children.
  - Only use the adapter provided with this product. Do not use other adapters. Do not use the breast pump's power adapter to power other devices.
  - If the power adapter is damaged, replace it with the one purchased from the manufacturer.
  - Do not attempt to repair the device yourself. The device can be repaired by the manufacturer's authorised service centre only.
  - Making any repairs or modifications to the product by unauthorised persons results in loss of warranty and may result in risks to health and life.
  - The manufacturer reserves the right to make design and other modifications to the device.
- NOTE! FAILURE TO FOLLOW THESE INSTRUCTIONS COULD RESULT IN INJURY OR PERMANENT DAMAGE TO YOUR PRODUCT.**
- Use the device only as intended and described in this user manual. Other uses violate the terms of the warranty.
  - Check the product before each use. Never use a product with any signs of damage. In case of damage, contact the manufacturer or distributor service.
  - It is forbidden to use the device if it has been dropped and fallen on a hard surface. In this case, the product shall be returned to the service centre for inspection.
  - Never use the device during the bath.
  - Never use the device in locations exposed to contact with water, for example near a bathtub, washbasin, kitchen sink or shower unit. DO NOT touch the device if it has fallen into water or other liquid. If the device is plugged into a power socket, unplug it immediately.

### INTRODUCTION

**The breast pump is equipped with a two-phase pumping system, stimulation and expression modes.** In each mode, you can choose one of 9 levels of suction. Additionally, the device is equipped with a bionic mode with 9 levels of suction that imitates a baby's suckling, a special flat nipple correction mode with 3-level adjustment and a nasal aspirator.

Breast milk is the healthiest and most valuable food for newborns and babies. It is recommended as the sole food given to your baby in the first six months of their life. After six months, it should be combined with solid foods.

**In case of physiological problems, consult your doctor immediately.**

#### Important safety information:

- Never use a breast pump during pregnancy as it can cause labour contractions.
- If you experience problems with breastfeeding or feeding your baby causes you pain you shall seek lactation consultant advice before using the breast pump.
- It is best to use the breast pump for the first time under specialist supervision.
- We recommend reading specialist literature and getting to know the principles of feeding a baby.

Make sure the breast pump is disassembled, washed and sterilised before use. Disassemble, wash and sterilise the parts of the breast pump that come into direct contact with breast milk, always before and after using the breast pump.

- Do not use the device while sleeping or in a state of excessive fatigue.
  - Do not immerse the control panel and power adapter in water. Do not wash them with a wet sponge or wet cloth.
  - Improper use may damage the device.
  - USE only indoors. DO NOT use outdoors.
  - This device is intended for household use only.
  - If the device has been transported at low temperatures, it should be acclimatized before use to equalize the appliance's temperature with the ambient conditions.
  - Make sure that the device is properly assembled before switching it on.
  - To stop the breast pump during the cycle, press the ON/OFF button again.
  - DO NOT use any caustic substances or chemicals for cleaning.
  - This product is not a toy. Use and store the device and all its parts out of the reach of children.
  - Do not disassemble the product.
  - DO NOT use power adapters that were not included with the unit.
  - Do not redesign or modify the product. Any action contrary to this guideline voids the EC declaration of conformity of the product.
  - If you have any doubts about the safety of the product, please contact with distributor or manufacturer.
- The information contained in this manual is subject to appropriate technical changes.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. This product and its information meet the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures.

The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class III appliance

This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information.

### DEVICE SPECIFICATION

Battery power supply Li-ion battery 3.7 V, 2500mAh	AC power supply 100-240 V / 50/60 Hz	Maximum power 10 W
--	--------------------------------------	--------------------

Do not wash or sterilise the control panel or power adapter.

- Keep the device clean. Prevent deposition of dust and other contaminants that could affect the performance of the product.
- Taking care of hygiene is essential for the safety of a mother and her baby.
- Parts of the device which cannot be washed should be protected against water and moisture.
- NOTE! Insufficient suction may result in a smaller amount of expressed milk, but too high suction can cause soreness during expression and reduce the production of milk. If you experience any of these problems, adjust the suction power.

#### When should I use a breast pump?

- If possible, wait until your lactation stabilises, which usually occurs within 2-4 weeks after birth. Exceptions to this rule:
  - Your breasts produce too much milk, and they become swollen and hard (breast fullness). Expressing a small amount of milk before or during feeding can give you relief and help your baby to grasp the nipple.
  - Your nipple is inverted because of the long interval between feedings.
  - You have concave, flat or damaged nipples (using a breast pump can improve nipple shape).
  - You feel overfluffed between feedings and do not want to wake the baby.
  - The baby is sick and does not want to suckle.
  - Your breasts produce too little milk, using the breast pump between feedings helps increase milk production.
- If you are working, or you are away from home and want to continue breastfeeding, you should regularly express milk using a breast pump to sustain the natural process of lactation.

#### How often should I express milk?

If you temporarily cannot breastfeed, it is recommended that you express milk using a breast pump at least eight times a day. That will keep your milk production at the appropriate level. In the beginning, it may be difficult for you to express more milk, but after a few days, you will be able to express more. Even with stable lactation, the possibility of expressing milk in one session is not big (30-60 ml), therefore to collect one portion of milk, you may require several expressing sessions.

### PARTS Fig. 1

a. Silicone massage cushion x2	g. Funnel cover x2	m. Screw ring x2
b. Breast pump body x2	h. Silicone tube x2	n. Sealing disk x2
c. Diaphragm x2	i. Tube connector x4	o. Bottle adapter x2
d. Cover x2	j. Pump unit with control panel	p. Power adapter
e. Silicone anti-return valve x2	k. Bottle cap x2	q. Charging cable with USB
f. Bottle x2	l. Teat x2	

Illustrations in this user manual are for reference only. Please refer to the actual product.  
The silicone anti-return valve, silicone massage cushion and tube are wear parts. If you notice wear and tear, purchase replacement parts from an authorised retailer.  
Wash the non-return valve very gently and do not allow food to dry inside it.

**NOTE!** Always remove the anti-return valve by carefully pushing its rim with your finger through the diaphragm cup. Never grasp the tip of the valve, as it can be damaged what can weaken the suction power.

#### CONTROL PANEL Fig. 2

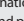
1. Nipple correction mode / nasal aspirator mode	7. Single or dual mode button	12. Level down
2. Level up	8. ON/OFF button	13. Battery indicator
3. Vacuum level indicator	9. Mode indicator (stimulation, expression, bionic)	14. Operating time indicator
4. Level down	10. Level up	15. Breast pump / aspirator mode
5. Suction frequency indicator (high, medium, low)	11. Vacuum level indicator	16. Frequency button
6. Mode button (stimulation, expression, bionic)		17. Pause

1. ON/OFF button : Press and hold for approximately 2 seconds to turn the device on or off.

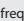
2. Mode change button:  Press the button once to change the mode.

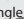
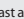
You can choose one of the following modes:

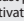
- Stimulation mode stimulates milk let-down and has 9 levels of suction. After running for 2 minutes in the stimulation mode, the breast pump will automatically switch to the expression mode at the first level of suction. You can change a mode to stimulation or expression at any time by pressing the button.
- The expression mode is designed specifically for milk expression. Also in the expression mode, you can choose one of 9 suction levels.
- Bionic mode imitates baby's sucking. It is a combination of stimulation and expression modes that work alternately, and it has 9 levels of strength.

3. Pause : Press the pause to stop the device, and press again to resume. The device turns off automatically after 3 minutes from pressing the pause.

4. Suction strength: You can press the "A" to increase the suction power or "V" to lower the suction power. You can adjust the suction power for each breast separately.

5. Frequency button : There are three levels of suction frequency available (low, medium and high frequency). The default value is medium frequency.

6. Single/double mode: Press the button  to switch between single breast and double breast modes. If you want to express milk from both breasts, connect the tube and accessory kit to the port marked with the icon  on the breast pump base housing.

7. Flat nipple correction mode: Effectively helps to stimulate concave nipples. To activate the flat nipple correction mode press the button . In this mode, you can adjust the suction level from 1 to 3.

#### ASSEMBLY Fig. 3

**NOTE!** Take special care when assembling and disassembling the breast pump to avoid damage to its components.

1. Place the silicone massage cushion (fig. 1a) in the funnel. Stretch its edges over the edges of the funnel (fig. 1b) so that they are fitted tightly.
2. Place the diaphragm (fig. 1c) in the cup at the top of the body (fig. 1b) so that they fit precisely. Press the edges of the diaphragm. Then place the diaphragm cap (fig. 1d).
3. Place the valve (fig. 1e) in the hole on the underside of the breast pump body (fig. 1b) by pushing it to the end. Screw the adapter onto the bottle (fig. 1f). Screw the whole set to the underside of the body (fig. 1b) so that they form one unit.
4. Connect one end of the tube (fig. 1h+i) to the top cover (fig. 1d) and the other end (fig. 1h+i) to the pump (fig. 1j). Make sure the tube is not tangled.
5. Place the funnel cover (fig. 1g). The device is ready.

**NOTE!** Before first use, fully charge the battery. The breast pump has a rechargeable lithium-ion battery and does not require AA batteries or a permanent connection to the mains. Once fully charged, stop charging to avoid damaging the battery. To extend the life of the battery, do not let it run down fully. The battery is charged via the power adapter. To charge the battery, connect one side of the charging cable to the breast pump and the other part to the power adapter. Then plug the power adapter into a power socket. It takes about **180 minutes** to fully charge the breast pump's battery. When the battery is fully charged, the battery LED indicator will stop flashing. The breast pump can be charged at any time. You don't have to wait for the battery to be fully discharged. On a full charge, the breast pump can run for **about 180 minutes**.  
The breast pump has a built-in timer and automatically switches off after 30 minutes of operation. If you want to restart the device, press the ON/OFF button.

#### USER MANUAL

Before each use:

- Wash your hands thoroughly.
- Make sure that the breast pump was pre-cleaned and dried. If not, follow the section Cleaning and Sterilising.
- Make sure that there is no dirt, liquid or moisture in the silicone tubing. Never turn on the breast pump if you notice water in the silicone tube. Warning! Do not mouth blow the silicone tube to remove water.

Place the breast pump on your breast so that the nipple is deep inside the funnel (exactly in the middle).

**NOTE!** The silicone cushion must cling tightly to the breast. Otherwise, the suction may be reduced.

1. Sit comfortably.
2. Make sure the batteries in the breast pump are charged.
3. Press the ON/OFF icon to start pumping. The breast pump will turn on to the 1 suction level in the stimulation mode. Then, after running for 2 minutes, it will automatically switch to the expression mode (first level). The breast pump is equipped with a timer to keep track of the length of the pumping session. The device automatically turns off after 30 minutes of use.
4. You can choose one of the available modes. After running for 2 minutes in the stimulation mode, the breast pump will automatically switch to the expression mode at the first level of suction.
5. After using, disassemble the breast pump. Clean the parts according to the Cleaning and Sterilising section.

**NOTE!** The breast pump, silicone tubes with connectors, power adapter and USB cable cannot be washed or sterilised in water. To clean, wipe with a cloth.

**NOTE!** Before putting the breast pump away from the breast, make sure it is turned off. If necessary, you can put the breast pump away during pumping by placing your finger between the pad and the breast (just as you do it when putting the child away).

#### CLEANING AND STERILISING

Disassemble the breast pump. Before first and after each use, wash all parts that come in direct contact with milk, diaphragm cover (fig. 1d), diaphragm (fig. 1c), breast pump body with funnel (fig. 1b), silicone massage cushion (fig. 1a), funnel cover (fig. 1g), silicone anti-return valve (fig. 1e), bottle adaptor (fig. 1o) and bottle (fig. 1f), in warm water with mild detergent and place them in boiling water for 5 minutes to ensure hygiene. Dry all components thoroughly before use. **NOTE!** Do not leave components on the bottom of the pot. Stir the water during sterilisation to prevent damage to components.

The silicone components in contact with the milk, i.e. the diaphragm (fig. 1c), silicone massage cushion (fig. 1a) and silicone anti-return valve (fig. 1e), can be sterilised in an electric steriliser.

How to disinfect the teat?

Before first and each use wash all parts of the product and disinfect. Place for 5 min in boiling water. This is to ensure hygiene. Clean before each use. Boiling and hard water may create lime scale effect. Wash all parts thoroughly with clean water, including areas that may be invisible but where food has passed, to remove any food residues. After using, remove the remaining food from the bottle and the teat, rinse, wash and disinfect in a steam/electric sterilizer or in boiling water. Bottle can be washed in dishwasher (max. temp. 60°C). It is not recommended to place the teat in a dishwasher. Hard chemicals / detergents may damage the product. Do not use any antibacterial agents. Inspect the feeding teat before each use and pull the feeding teat in all directions. Especially when the baby has teeth. Throw away at the first signs of damage or weakness. Do not tighten up the bottle locking ring, as it may disturb the teat's ventilation. Always check for leakage. Never enlarge the teat flow holes by any means, as it can damage the teat. Take particular care when heating in a microwave. Always stir heated food to ensure even distribution of temperature and check liquid temperature before giving it to the child. When warming food in a microwave, remember to leave the bottle open. Do not place product in hot oven or heat in a gas/electric cooker. Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. It is recommended to replace a teat max. after 2 months of use. Retain the instruction for later use. Bottle made from Polypropylene. Batch code is placed on the packaging.

**For your child's safety and health**

**WARNING!**

Continuous and prolonged sucking of fluids will cause tooth decay.

Always check food temperature before feeding.

Throw away at the first signs of damage or weakness.

Keep components not in use out of the reach of children.

Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.

Never use feeding teats as a soother.

Always use this product with adult supervision.

**WARNING!**

• Do not use this product for purposes other than those listed in this user manual.

• Do not place the product near sources of fire.

• Before using the breast pump, make sure that all parts were assembled correctly.

• If you feel breast or nipple discomfort, stop using the breast pump immediately.

• Wash your hands before using the product.

• Keep the product out of the reach of newborns and children.

• Do not drop or hit the product to avoid damage.

• Do not touch product elements with bare hands immediately after sterilisation in boiling water to avoid burns.

Do not resell a used device.

#### MILK STORAGE

1. Due to the active biochemical and microbiological processes occurring in the milk, it is important to maintain a constant temperature of storage.
2. Expressed milk store in bottles, special sterile bags or sealed containers with a lid (which should be disinfected before use).
3. Mark each container with the date.
4. Freeze only fresh milk. Do not fill a container fully as frozen breast milk increases its volume.
5. Breast milk can be heated in the same way as any other food (but never in the microwave as this may destroy valuable nutrients and antibodies).
6. Frozen breast milk can be defrosted overnight in the refrigerator and then warmed gradually in a bottle warmer or hot water.
7. After defrosting, shake the container or stir the milk to mix the separated fat particles with the remaining volume of milk.
8. DO NOT store milk on shelves placed on a refrigerator door.
9. NEVER refreeze previously defrosted milk.
10. DO NOT add fresh breast milk to defrosted milk.
11. DO NOT add fresh breast milk to frozen milk.
12. NEVER heat breast milk by boiling or putting it in the microwave.
13. ALWAYS check the liquid temperature before feeding.
14. Dispose of any milk that has not been used.

Type of breast milk	Temperature (°C)	Recommended time of storage
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum 24 hours
	+19°C to +25°C (room temperature)	optimally up to 4 hours
Frozen milk	from +25°C to +37°C	maximum up to 4 hours
Thawed milk	-20°C	optimally 3-6 months
	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	up to 4 hours

## TROUBLESHOOTING

When using any breast pump, you may encounter minor problems. Before you decide to place the claim, try to solve these problems by yourself. Here are some useful tips.

### Pain when using the breast pump

- Put a warm compress on your breast and wait 5 minutes. Then use the breast pump again.
- Increase the suction level gradually, each time by 1 level.
- If the above steps do not resolve the problem, stop using the breast pump and consult your doctor.

### The breast pump doesn't work

- Check if the battery is charged.
- If the breast pump still does not work contact the seller or distributor.

### The suction power is too low or zero

- If you observe a weakening of the suction power, do the following.
- Ensure that the breast pump has been properly assembled and that all connections between components are made correctly.
- Check if the valve is placed tightly in the body and is not damaged. This component is crucial to achieving adequate suction power. If it has been damaged, replace it with a new one.
- Check if the breast fits tightly into the body with the funnel.

### Milk gathers in the silicone massage cushion

- Make sure that you are holding the breast pump correctly. Keep the bottle in a vertical position.

### Backflow of milk

- If milk or water finds its way into the silicone diaphragm or tube, clean it according to the instructions. Remember to dry all parts. Check if the valve is not damaged.

The breast pump is intended for personal use. It is not refundable or exchangeable. It is covered by the manufacturer's warranty. Remember that the proof of purchase is the basis to examine the complaint.

Remember that if you have lactation problems as a result of which your baby is unable to empty the breast, a breast pump may also be ineffective.

## ASPIRATOR

**Usage:** Use the device to remove nasal secretions. Do not use for other purposes. Recommended for infants under 6 months.

To use the nasal aspirator mode, replace suction accessories.

**Warning!** To avoid damage to the pump unit, make sure the suction accessories are dry. When removing secretions, ensure that the amount of secretion sucked does not exceed the capacity of the secretion container.

### Aspirator parts fig. 4

1. Cover
  2. Suction tip, 2 pcs
- We recommend using the narrow tip when the mucus has a liquid form.  
We recommend using the wide tip when the mucus has a more solid form.
3. Aspirator housing
  4. Secretion container
  5. Aspirator base
  6. Silicone tube with connectors

### HOW TO USE:

1. Place the aspirator housing (fig. 4.3) on the secretion container (fig. 4.4).
2. Insert the assembled component into the aspirator base (fig. 4.5) and screw them tightly.
3. Place the chosen suction tip, narrow or wide (fig. 4.2), onto the assembled aspirator housing.
4. Connect the aspirator with the silicone tube (fig. 4.6) to the breast pump.

Turn the device on by pressing the ON/OFF button. Then use the button to select the aspirator mode. You can adjust the suction power by choosing one from 9 available suction levels. You shall suck secretion from one nostril for about 5 seconds.

**NOTE!!!** There is a special hole in the housing – a vent (fig. 5). Cover it with your finger during use to ensure adequate suction power.

### AFTER USE

1. Disconnect the aspirator from the pump and then disassemble.
2. Remove the collected secretions from the container.
3. Clean aspirator parts thoroughly under running water with a mild detergent.
4. Thoroughly dry all parts after washing.
5. Put the protective cover (fig. 4.1) on.

### NOTE:

1. The nasal aspirator is not a toy. It can be used by an adult only.
2. Keep out of reach of children. Potential choking hazard.
3. DO NOT insert too deeply into a nostril. Do not press the tip against the walls of a nose.
4. Check the condition of the device and its components regularly.
5. Use the device only as intended and described in this user manual. Never use on ears, mouth, or eyes.
6. Keep the device cleaned and ready for use.

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## DECLARATION OF CONFORMITY EU No. 4/2022

In scope of directives: 2014/35/EU, 2014/30/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o.  
Puławska Street 430  
02-884 Warsaw  
Poland

We hereby declare under our responsibility that **double electric breast pump ExpressCare with power supply** cat. no. (model) 12/212  
brand **Canpol babies**

is in accordance with the following directives:

- Directive relating to the making available on the market of electrical equipment designed for use within certain voltage limits 2014/35/EU
- Directive relating to electromagnetic compatibility 2014/30/EU
- Energy related Products Directive 2009/125/EC
- RoHS Directive 2011/65/EU and 2015/863/EU

and is in accordance with the following harmonized standards:

EN 60335-1:2012 + A11:2014 + A13:2017  
EN 55014-1:2017  
EN 55014-2:2015  
EN 61000-3-2:2014  
EN 61000-3-3:2013  
EN 62233:2008  
EN IEC 63000:2018

as well as other technician specifications:

The Commission Regulation (EU) No. 2019/1782

The last two digits of the year in which the **CE** marking was affixed: 22

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Stanisława Piąteczkowska  
Quality Assurance Manager

(name, surname and signature of authorized person,  
position)

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